

Test Sequence

Infusion Device – IEC 60601-2-24 (v1.0)

Template	Infusion Device – IEC 60601-2-24
Version	1.0
Created	2025-10-02 01:08
Created By	Seed

Test Steps

#	Step / Instruction	Type	Limits	Unit	Actual / Pass
1	Flow Rate Accuracy (100 ml/h) Set 100 ml/h; measure delivered rate. Ref: IEC 60601-2-24 §201.12.1.101	Measurement	100 ml/h ±5%	ml/h	_____
2	Occlusion Alarm Pressure Raise line pressure until alarm; record threshold. Ref: IEC 60601-2-24 §201.12.4	Measurement	500 mmHg ±100	mmHg	_____